

Checklist - Stool Collection Package

Complete the checklist to ensure you have all items in your package.

Your package includes:

- One OMNIgene-GUT Stool Collection Kit **(A)**
- One OMNImet-GUT Stool Collection Kit **(B)**
- One toilet accessory (stool hat) to place on your toilet bowl **(C)**
- A pair of latex-free disposable gloves **(D)** **if you wish to use them*
- Two alcohol pads **(E)**
- One biohazard plastic bag with two labels **(F)**
 - One for date and time
 - One to complete after mixing both tubes (with a picture)
- One ice pack **(G)**
- One thermal envelope **(H)**



If your package is not complete, please contact us.

Instructions for Stool Collection

PART 1: THE DAY YOU RECEIVE YOUR KIT

- Complete the content checklist to ensure you have all items.
- Store the ice pack in your freezer.

PART 2: THE DAY YOU COLLECT YOUR STOOL SAMPLE

Wait until your Data Collection Site visit is booked before collecting the sample. Take the sample no more than 4 days before your visit to the Data Collection Site.

STEP 1: READ instructions for

- Collecting the stool specimen: See Instructions for Use – Stool Hat
- Using the collection tubes: See **OMNImet-GUT tube** (with red illustrations)

*Note: instructions are identical for the **OMNIgene-GUT tube** instructions (with purple illustrations)*

STEP 2: COLLECT stool into tubes.

***Use sample from only one bowel movement for both tubes.**

***SHAKE the sealed tubes VIGOROUSLY for 30 seconds or longer to mix the sample with the preservative.**

Clean the exterior of the tubes with the alcohol pad.

STEP 3: RECORD information on the biohazard bag (**white labels**) using a pen:

- Date and time of sample collection.
- Check the box confirming BOTH tubes have been well mixed.

STEP 4: PACKAGE the tubes.

Place the two tubes in the biohazard bag and seal.

Insert the biohazard bag into the thermal envelope. **DO NOT SEAL.**

Store the envelope **at room temperature**. **DO NOT FREEZE.**

PART 3: THE DAY OF YOUR DATA COLLECTION SITE VISIT

- Place the ice pack into the thermal envelope and seal it.
- Take the thermal bag with you to your visit.

If you require assistance, please contact us:



1-866-999-8303



www.clsa-elcv.ca



info@clsa.elcv.ca